



## CELOX™ VASCULAR HEMOSTATIC PAD

Now available to you for use on your post-catheterization patients when pulling sheaths, the new **CELOX™ Vascular** hemostatic Pad is used in conjunction with manual or mechanical compression, in generally the same way that you use other hemostatic dressings.

CELOX's unique, patented and natural formula works independently of the body's normal clotting mechanisms to clot blood even where normal clotting is slow or impaired. In clinical testing CELOX has repeatedly shown itself to quickly and reliably stop even deep arterial bleeds. CELOX Vascular includes the same chitosan that has already been shown to result in 100% survival in independent US Marine Corps' bleeding tests (data on file).

CELOX Vascular is indicated for the local management and control of surface bleeding from vascular access sites, percutaneous catheters or tubes utilizing introducer sheaths up to 16 French".

*Contains chitosan. Sterile, single use only.*

*For additional information, please contact:*

Advanced Vascular Dynamics  
Phone: (800) 525-2555 Fax: (503) 223-8585

## Instructions for Use

1. Remove CELOX Vascular Pad from sterile package.
2. Place CELOX Vascular Pad over the access site and point of compression.
3. Remove sheath, ensuring some blood touches the CELOX Vascular Pad. Apply firm manual compression until hemostasis is achieved.
4. Leave CELOX Vascular Pad in place. Apply adhesive bandage over Pad if required.
5. CELOX Vascular Pad may be left in place for up to 24 hours.
6. To remove, gently peel away the adhesive bandage (if used). Next, gently remove the CELOX Vascular Pad, using saline if required.



[www.compressor.com](http://www.compressor.com)